

# Manage postpartum hemorrhage.




**Bakri®**

POSTPARTUM BALLOON WITH RAPID INSTILLATION COMPONENTS



# A simple solution for postpartum hemorrhage (PPH)

## Sequential treatment considerations for conservative management of uterotonic-resistant PPH<sup>1</sup>

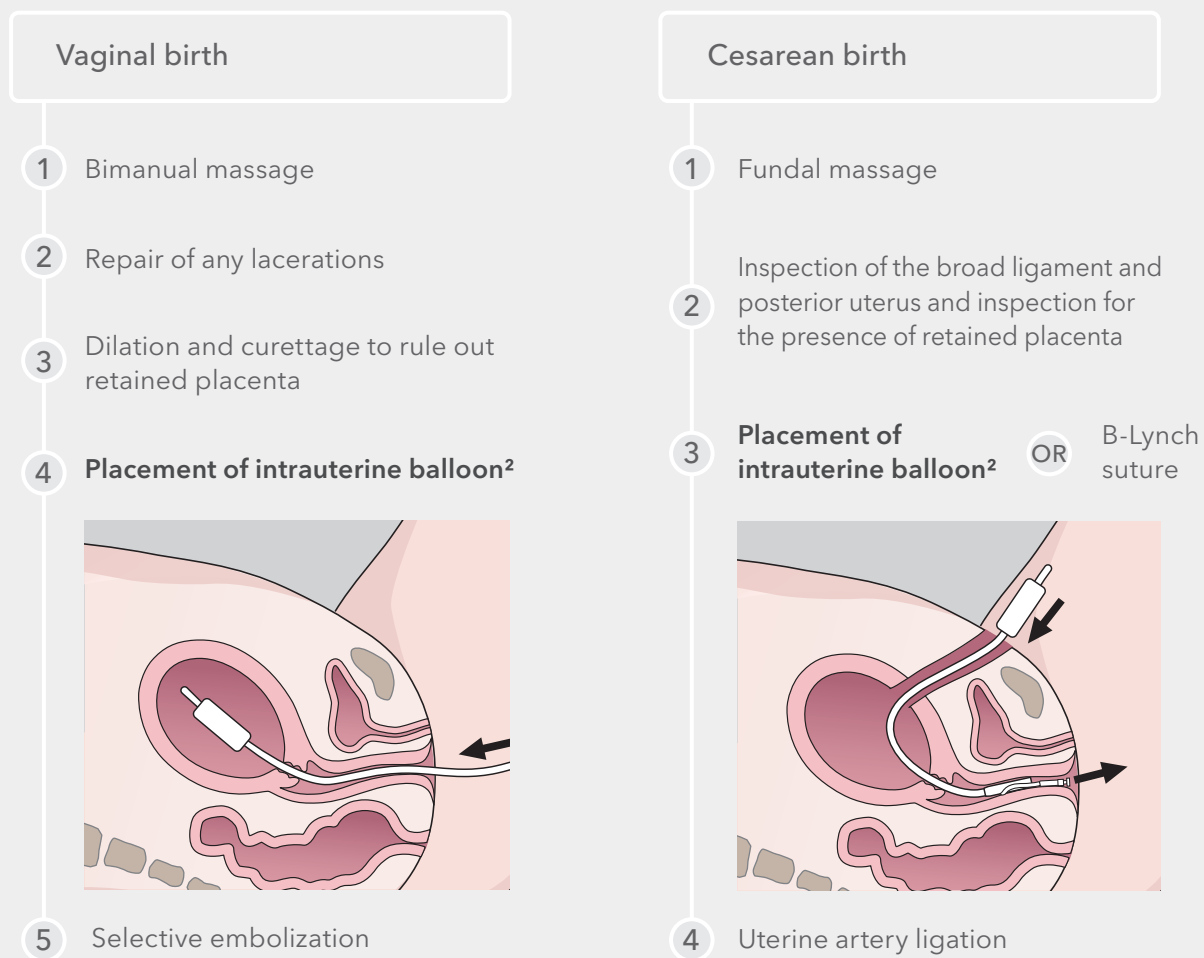

Note: The above flowchart describes sequential treatment considerations for uterotonic-resistant PPH when blood loss is less than 1,500 mL and after evaluation for uterine inversion and amniotic fluid embolism.

1. California Maternal Quality Care Collaborative. Obstetric hemorrhage emergency management plan: table chart format. Version 2.0. Published 2014. Accessed August 31, 2016.

2. Please refer to the IFU for detailed indications, contraindications, warnings, and precautions.

The Bakri Postpartum Balloon with Rapid Instillation Components is used for temporary control or reduction of postpartum uterine bleeding when conservative management of uterine bleeding is warranted.



When inflated with sterile liquid, the Bakri Postpartum Balloon with Rapid Instillation Components applies pressure to the uterine walls that may provide management of postpartum hemorrhage.

The balloon and catheter are constructed of 100% silicone.

- Can be used after vaginal or cesarean delivery
- Includes rapid instillation components to facilitate inflation of the balloon
- May be used with B-Lynch compression sutures if clinically warranted






## Bakri®


POSTPARTUM BALLOON WITH RAPID INSTILLATION COMPONENTS

This device is intended to provide temporary control or reduction of postpartum uterine bleeding when conservative management is warranted. The set includes a syringe, 180 cm long extension tubing with bag spike, and dual check valve. Please refer to the Instructions for Use for complete information on product usage, indications, and contraindications.

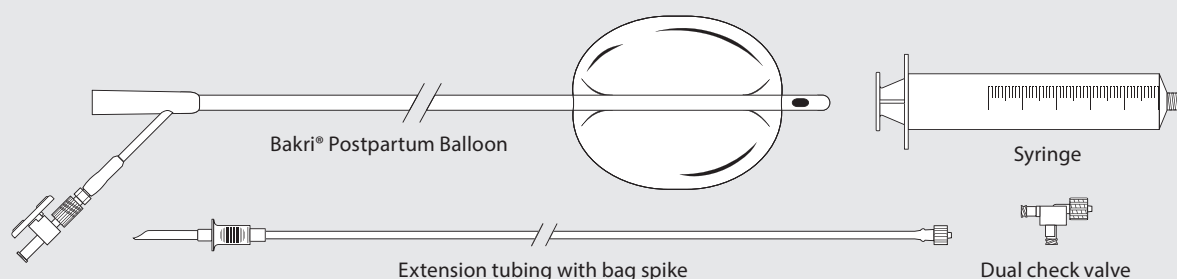

| Order Number | Reference Part Number | Catheter Fr | Length cm | Balloon Volume mL |
|--------------|-----------------------|-------------|-----------|-------------------|
| G24237       | J-SOSR-100500         | 24          | 54        | 500               |

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

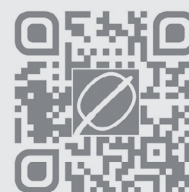

Scan this QR code to see other resources.

### Resources

- Georgiou C. Intraluminal pressure readings during the establishment of a positive 'tamponade test' in the management of postpartum haemorrhage. *BJOG*. 2010;117(3):295-303.
- Vintejou E, Ulrich D, Mousty E, et al. Success factors for Bakri™ balloon usage secondary to uterine atony: a retrospective, multicentre study. *Aust N Z J Obstet Gynaecol*. 2015;55(6):572-577.
- Vithala S, Tsoumpou I, Anjum ZK, et al. Use of Bakri balloon in postpartum haemorrhage: a series of 15 cases. *Aust N Z J Obstet Gynaecol*. 2009;49(2):191-194.

### Customer Service

EU Website: [cookmedical.eu](http://cookmedical.eu)  
 EDI: [cookmedical.eu/edi](http://cookmedical.eu/edi)  
 Distributors: +353 61239240, [ssc.distributors@cookmedical.com](mailto:ssc.distributors@cookmedical.com)  
 Austria: +43 179567121, [oe.orders@cookmedical.com](mailto:oe.orders@cookmedical.com)  
 Belgium: +32 27001702, [be.orders@cookmedical.com](mailto:be.orders@cookmedical.com)  
 Denmark: +45 38487607, [da.orders@cookmedical.com](mailto:da.orders@cookmedical.com)  
 Finland: +358 972519996, [fi.orders@cookmedical.com](mailto:fi.orders@cookmedical.com)  
 France: +33 171230269, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)  
 Germany: +49 6950072804, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)  
 Hungary: +36 17779199, [hu.orders@cookmedical.com](mailto:hu.orders@cookmedical.com)  
 Iceland: +354 800 7615, [is.orders@cookmedical.com](mailto:is.orders@cookmedical.com)  
 Ireland: +353 61239252, [ie.orders@cookmedical.com](mailto:ie.orders@cookmedical.com)  
 Italy: +39 0269682853, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)  
 Netherlands: +31 202013367, [nl.orders@cookmedical.com](mailto:nl.orders@cookmedical.com)  
 Norway: +47 23162968, [no.orders@cookmedical.com](mailto:no.orders@cookmedical.com)  
 Spain: +34 912702691, [es.orders@cookmedical.com](mailto:es.orders@cookmedical.com)  
 Sweden: +46 858769468, [se.orders@cookmedical.com](mailto:se.orders@cookmedical.com)  
 Switzerland - French: +41 448009609, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)  
 Switzerland - Italian: +41 448009609, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)  
 Switzerland - German: +41 448009609, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)  
 United Kingdom: +44 2073654183, [uk.orders@cookmedical.com](mailto:uk.orders@cookmedical.com)

USA Website: [cookmedical.com](http://cookmedical.com)

EDI: [cookmedical.com/edi.do](http://cookmedical.com/edi.do)

#### Americas:

Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335  
 E-mail: [customersupport@cookmedical.com](mailto:customersupport@cookmedical.com)

#### Australia:

Phone: +61 734346000, 1800777222, Fax: +61 734346001, 1800077283  
 E-mail: [cau.custserv@cookmedical.com](mailto:cau.custserv@cookmedical.com)



AI-ESC-IR-OHNS-PI-RH-SUR-A4