



Otologic Drills and Burs

REVOLUTIONARY DESIGN FOR OTOTOLOGY AND NEUROTOLOGY



How to Use This Book

Conveniently arranged by drill type, this brochure can help you find the recommended otologic bur in the Xomed collection. Review the descriptions below, then delve into the tabbed section for your handpiece.



① Indigo™

The new Indigo High-Speed Otologic Drill was designed for general otologic procedures. With its straight and angled attachments, it offers flexibility, excellent surgical access and visibility. Its small, lightweight body provides power and performance.

The Indigo drill is powered by the IPC system. It works with a wide variety of burs in tool steel, stainless steel and carbide. Operating up to 60,000 rpm, Indigo is the lightest drill in our ENT portfolio with the highest torque.

See pages 02-08.



⑤ Visao®

Designed specifically for neurotologists, the Visao High-Speed Otologic Drill allows use of very slim curved burs, ideal for cochleostomies, middle fossa acoustic neuromas, and infralabyrinthine approaches. Running up to 80,000 rpm, the water-cooled body of the motor helps reduce the temperature of the Visao drill.

See pages 09-16.



⑤ Skeeter®

The Skeeter Ultra-Lite Oto-Tool handpiece is a unique low-speed drill designed for delicate ossicular chain reconstruction procedures, at speeds up to 16,000 rpm.

See pages 24-25.



Tool Steel

Sharp every time. Carbon and alloy steel well-suited for otology due to distinctive hardness and resistance to abrasion.



Stainless Steel

Sharp edge and durability. Made with a 440-series stainless steel and coated with titanium nitride.



Carbide

Stiffer and denser for greater durability. Tungsten carbide tools are used to withstand high temperatures and for situations where other materials would wear away.



Oto-Flex Burs

Fine, malleable burs designed for the delicate work of ossicular chain reconstruction.



Curved Burs

Maximum visibility and access to the surgical field. Each bur has a 20° curve to improve visibility. The outer sheath shields the patient from the spinning shaft and the evaporative cooling sleeve reduces the distal bur sheath temperature.



Coarse Diamond

A cutting tool with fixed diamond grains. Diamond is a superhard material that allows cutting in all directions.



Fine Diamond

Finer diamond grains allow better sculpting possibilities and are less aggressive.



Length

Listings indicate total length of the bur, measured from base to tip.

Note: Packaging may reflect shaft length rather than total length.

I

V

S

Indigo™

High-Speed Otologic Drill

The new Indigo High-Speed Otologic Drill offers excellent surgical access and visibility. Its small, lightweight body provides the power and performance required for a wide range of otologic procedures.

Indigo Components

Use at speeds up to 60,000 rpm.

PRODUCT	DESCRIPTION	QTY
1845000	Indigo High-Speed Otologic Drill	1
1845010	Indigo Otologic Straight Attachment	1
1845020	Indigo Otologic Angled Attachment	1
1845040	Indigo Instrument Tray	1
1845030	Indigo Tubing	5



1845000



1845010



1845020



1845030

Indigo Drill System Key Features:

- ① High speed and ample torque for fast debulking of the mastoid
- ② Available with straight and angled attachments
- ③ Improved surgical access and visibility in cramped otologic spaces
- ④ Works with a wide variety of Medtronic Xomed otology burs
- ⑤ Grooves on attachment nose for more stable irrigation
- ⑥ Powered by the IPC® System with easy touchscreen menus




IndigoTM

Cutting Burs

Round Fluted Burs

7.0 – 3.5 mm Diameter

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	MATERIAL	PART NUMBER	/ BOX
	7.0	64	Tool steel*	31317057U	5
	6.0	64	Tool steel*	31316058U	5
	5.5	69	Stainless steel	31145564A	5
	5.0	64	Tool steel*	31315059U	5
	5.0	66	Tool steel*	31315066U	5
	5.0	68	Carbide [†]	3127655	1
	4.8	68	Stainless steel	31144864A	5
	4.0	67	Carbide [†]	3127654	1
	4.0	68	Stainless steel	31144064AE	1
	4.0	68	Stainless steel	9789B	5
	4.0	69	Tool steel*	31314065U	5
	4.0	72	Tool steel*	31314068U	5
	4.0	78	Tool steel*	31314075UE	1
	3.5	68	Tool steel*	31313565UE	1
	3.5	71	Tool steel*	31313568U	5
	3.5	78	Tool steel*	31313575UE	1

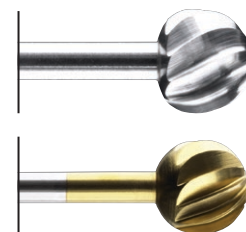

*Single use only

[†]Use at speeds up to 52,000 rpm


IndigoTM

Cutting Burs

Round Fluted Burs

3.0 – 2.0 mm Diameter

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	MATERIAL	PART NUMBER	/ BOX
●	3.0	66	Stainless steel	3112964C	5
●	3.0	66	Carbide [†]	3127653	1
●	3.0	68	Tool steel*	31313065UE	1
●	3.0	72	Tool steel*	31313069U	5
●	3.0	72	Stainless steel	9769B	5
●	3.0	79	Stainless steel	3112974C	5
●	3.0	79	Carbide [†]	3127754	1
●	2.5	67	Tool steel*	31312565UE	1
●	2.5	72	Tool steel*	31312569	1
●	2.5	77	Tool steel*	31312575	1
●	2.3	66	Stainless steel	31142364	5
●	2.0	65	Stainless steel	31142064	5
●	2.0	65	Carbide [†]	31062064	1
●	2.0	67	Tool steel*	31312065	1
●	2.0	72	Tool steel*	31312070	1
●	2.0	77	Tool steel*	31312075	1
●	2.0	78	Stainless steel	31142075	5
●	2.0	78	Carbide [†]	31062075	1

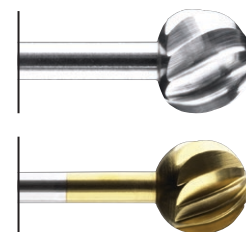

*Single use only

[†]Use at speeds up to 52,000 rpm


IndigoTM

Cutting Burs

Round Fluted Burs

1.5 – 0.5 mm Diameter

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	MATERIAL	PART NUMBER	/ BOX
•	1.5	65	Stainless steel	31141564	5
•	1.5	67	Tool steel*	31311565	1
•	1.5	72	Tool steel*	31311570	1
•	1.5	76	Tool steel*	31311575	1
•	1.5	77	Carbide [†]	31061575	1
•	1.5	78	Stainless steel	31141575	5
•	1.0	64	Stainless steel	31141064	5
•	1.0	64	Carbide [†]	31061064	1
•	1.0	67	Tool steel*	31311066	1
•	1.0	72	Tool steel*	31311071	1
•	1.0	76	Tool steel*	31311075	1
•	1.0	77	Stainless steel	31141075	5
•	1.0	77	Carbide [†]	31061075	1
•	0.5	64	Stainless steel	31140564	5
•	0.5	64	Carbide [†]	31060564	1

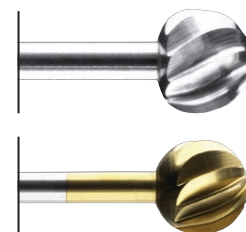

*Single use only

[†]Use at speeds up to 52,000 rpm


IndigoTM

Cutting Burs

Acorn Fluted Burs

5.0 – 1.0 mm Diameter

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	MATERIAL	PART NUMBER	/ BOX
	5.0	69	Carbide [†]	3121518	1
	4.6	68	Stainless steel	31034664	5
	3.7	67	Stainless steel	31033764	5
	3.0	67	Carbide [†]	3121516	1
	2.7	66	Stainless steel	31032764	5
	2.7	66	Carbide [†]	31012764	1
	2.3	66	Stainless steel	31032364	5
	2.3	66	Carbide [†]	31012364	1
	1.4	65	Stainless steel	31031464	5
	1.4	65	Carbide [†]	31011464	1
	1.0	65	Stainless steel	31031064	5
	1.0	65	Carbide [†]	31011064	1

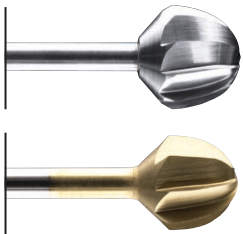

**Single use only*
†Use at speeds up to 52,000 rpm

IndigoTM

Diamond Burs

Round Coarse Diamond Burs

6.0 – 2.0 mm Diameter

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	MATERIAL	PART NUMBER	/ BOX
	6.0	64	Stainless steel	31116058	1
	5.0	64	Stainless steel	31115059	1
	5.0	71	Stainless steel	31115066	1
	4.0	69	Stainless steel	31114065	1
	4.0	72	Stainless steel	31114068	1
	4.0	78	Stainless steel	31114075	1
	3.5	68	Stainless steel	31113565	1
	3.5	71	Stainless steel	31113568	1
	3.0	66	Stainless steel	31113064	1
	3.0	68	Stainless steel	31113065	1
	3.0	72	Stainless steel	31113069	1
	3.0	78	Stainless steel	31113075	1
	2.5	67	Stainless steel	31112565	1
	2.5	71	Stainless steel	31112569	1
	2.5	77	Stainless steel	31112575	1
	2.0	67	Stainless steel	31112065	1
	2.0	72	Stainless steel	31112070	1
	2.0	77	Stainless steel	31112075	1



IndigoTM

Diamond Burs

Round Coarse Diamond Burs

1.5 – 1.0 mm Diameter

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	MATERIAL	PART NUMBER	/ BOX
●	1.5	67	Stainless steel	31111565	1
●	1.5	71	Stainless steel	31111570	1
●	1.5	76	Stainless steel	31111575	1
●	1.0	67	Stainless steel	31111066	1
●	1.0	72	Stainless steel	31111071	1
●	1.0	76	Stainless steel	31111075	1



IndigoTM

Diamond Burs

Round Fine Diamond Burs

6.0 – 2.0 mm Diameter

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	MATERIAL	PART NUMBER	/ BOX
	6.0	64	Stainless steel	31136058	1
	6.0	71	Stainless steel	31136065	1
	5.0	64	Stainless steel	31135059	1
	5.0	71	Stainless steel	31135066	1
	4.0	69	Stainless steel	31134065	1
	4.0	72	Stainless steel	31134068	1
	3.5	68	Stainless steel	31133565	1
	3.5	71	Stainless steel	31133568	1
	3.0	68	Stainless steel	31133065	1
	3.0	72	Stainless steel	31133069	1
	3.0	79	Stainless steel	31103075	1
	2.5	67	Stainless steel	31132565	1
	2.5	71	Stainless steel	31132569	1
	2.5	78	Stainless steel	31102575	1
	2.0	67	Stainless steel	31132065	1
	2.0	72	Stainless steel	31132070	1
	2.0	78	Stainless steel	31102075	1



IndigoTM

Diamond Burs

Round Fine Diamond Burs

1.5 – 0.5 mm Diameter

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	MATERIAL	PART NUMBER	/ BOX
●	1.5	67	Stainless steel	31131565	1
●	1.5	71	Stainless steel	31131570	1
●	1.5	77	Stainless steel	31101575	1
●	1.0	67	Stainless steel	31131066	1
●	1.0	72	Stainless steel	31131071	1
●	1.0	77	Stainless steel	31101075	1
●	0.5	67	Stainless steel	31130566	1
●	0.5	72	Stainless steel	31130571	1
●	0.5	77	Stainless steel	31100575	1



Visao[®]

High-Speed Otologic Drill

The Visao High-Speed Otologic Drill allows remarkable surgical visibility and access with patent-pending curved burs. This makes it ideal for cochleostomies, middle fossa acoustic neuromas, and infralabyrinthine approaches to the petrous apex.

Visao Components

Use at speeds up to 80,000 rpm.

PRODUCT	DESCRIPTION	QTY
3334800	Visao [®] High-Speed Otologic Drill	1
3334895	Visao Otologic Instrument Tray	1
3334625	Extended Bur Guard without Irrigation	1
3334635	Extended Bur Guard with Irrigation	1
3334610	Irrigation Sleeve Assembly	1
3318603	Irrigation Tubing with IntelliFlow™ Remote Control and Disposable Bur Guard	1
3318606	Visao Coolant Tubing with Cassette	2
3318503	Irrigation Tubing Set, Sterile	4
3318506	High-Speed Drill Coolant Tubing Set	2
3318600	Clips for Handpiece Cable/Tubing	10
3318700	Irrigation Tubing Retainer	1

Visao Drill System Key Features:

- ① Powered by the IPC[®] System with easy touchscreen menus
- ② Utilizes snap-fit, reusable bur guards and quick-locking burs ideal for busy teaching institutions
- ③ Lightweight titanium provides a well-balanced, ergonomic handpiece
- ④ Virtually no start-up “kick” with low operating temperatures and integrated water-cooling
- ⑤ 80,000 rpm and high torque capabilities provide power in debulking operations and finesse in delicate procedures
- ⑥ High-speed curved burs of various sizes help provide better visibility and access in several procedures



Visao[®]

Curved High-Speed Burs

Curved Cochleostomy Burs*

Round Fluted Cutting Tip

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	MATERIAL	PART NUMBER	/ BOX
●	1.5	100	Stainless steel	31151598E	1
●	1.0	99	Stainless steel	31151098E	1



Curved Cochleostomy Burs*

Round Fine Diamond Tip

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	MATERIAL	PART NUMBER	/ BOX
●	1.5	100	Stainless steel	31181598E	1
●	1.0	99	Stainless steel	31181098E	1
●	0.6	99	Stainless steel	31180698E	1
●	0.5	99	Stainless steel	31180598E	1



Curved Burs Key Features:

- ① Revolutionary 20° curved burs allow better visibility and access to the surgical field
- ② Cochleostomy burs are thinner and tapered at the tip for greater visibility
- ③ Useful during complex procedures with only small access holes or speculums
- ④ Thin outer sheaths leave very little rotating shaft exposed, helping to protect delicate tissue from a spinning shaft
- ⑤ Use at speeds up to 80,000 rpm, forward or backward



Visao[®]

Curved High-Speed Burs

Curved Round Fluted Cutting Burs*

4.0 – 1.0 mm Diameter

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	MATERIAL	PART NUMBER	/ BOX
	4.0	94	Stainless steel	31154090E	1
	4.0	119	Stainless steel	311740115E	1
	3.0	93	Stainless steel	31153090E	1
	3.0	118	Stainless steel	311730115E	1
	2.0	92	Stainless steel	31152090E	1
	2.0	117	Stainless steel	311720115E	1
	1.5	117	Stainless steel	311715115E	1
	1.0	116	Stainless steel	311710115E	1



**Single use only*

Visao[®]

Curved High-Speed Burs

Curved Round Fine Diamond Burs*

4.0 – 1.0 mm Diameter

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	MATERIAL	PART NUMBER	/ BOX
	4.0	94	Stainless steel	31184090E	1
	4.0	119	Stainless steel	311940115E	1
	3.0	93	Stainless steel	31183090E	1
	3.0	118	Stainless steel	311930115E	1
	2.0	92	Stainless steel	31182090E	1
	2.0	117	Stainless steel	311920115E	1
	1.0	116	Stainless steel	311910115E	1



Curved Round Coarse Diamond Burs*

4.0 – 2.0 mm Diameter

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	MATERIAL	PART NUMBER	/ BOX
	4.0	94	Stainless steel	31204090E	1
	4.0	119	Stainless steel	312140115E	1
	3.0	93	Stainless steel	31203090E	1
	3.0	118	Stainless steel	312130115E	1
	2.0	92	Stainless steel	31202090E	1
	2.0	117	Stainless steel	312120115E	1




**Single use only*

Visao®

Stim Bur Guards

Stim Bur Guard Components

PRODUCT	DESCRIPTION	QTY
3318601	Stim Bur Guard with Irrigation	1
3318602	Stim Bur Guard without Irrigation	1

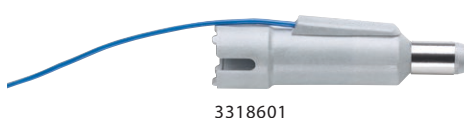

3318601



3318602

Stim Bur Guard Key Features:

- ① The Stim Bur Guard emits stimulation generated by our NIM® nerve monitors to provide early warning of the bur's proximity to the facial nerve. Stimulating the nerve while the bur is in use can offer approximately 1-3 mm of advance warning and provides valuable information during otologic surgery.
- ② By reducing the risk of patient injury, the innovative Stim Bur Guard helps provide additional peace-of-mind.
- ③ Stim bur guards must be used with the following products:
Integrated Power Console (IPC®) and Visao® High-Speed Otologic Drill
- ④ Stim bur guards are also compatible with the following products:
NIM-Neuro® 3.0, NIM-Response® 3.0, NIM-Pulse® 3.0, NIM-Neuro® 2.0, NIM-Response® 2.0, or NIM-Pulse® 2.0




Visao[®]

Cutting Burs

Round Fluted Burs

7.0 – 1.0 mm Diameter

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	MATERIAL	PART NUMBER	/ BOX
	7.0	64	Tool steel*	31317057U	5
	7.0	69	Tool steel*	31317062U	5
	6.0	64	Tool steel*	31316058U	5
	6.0	69	Tool steel*	31316063U	5
	5.0	64	Tool steel*	31315059U	5
	5.0	66	Tool steel*	31315066U	5
	4.0	68	Stainless steel	9789B	5
	4.0	69	Tool steel*	31314065U	5
	4.0	72	Tool steel*	<u>31314068U</u>	5
	3.5	71	Tool steel*	<u>31313568U</u>	5
	3.0	72	Tool steel*	<u>31313069U</u>	5
	3.0	72	Stainless steel	<u>9769B</u>	5
	3.0	79	Stainless steel	<u>3112974C</u>	5
	2.5	72	Tool steel*	<u>31312569</u>	1
	2.0	72	Tool steel*	<u>31312070</u>	1
	2.0	78	Stainless steel	<u>31142075</u>	5
	1.5	72	Tool steel*	<u>31311570</u>	1
	1.5	78	Stainless steel	<u>31141575</u>	5
	1.0	72	Tool steel*	<u>31311071</u>	1
	1.0	77	Stainless steel	<u>31141075</u>	5

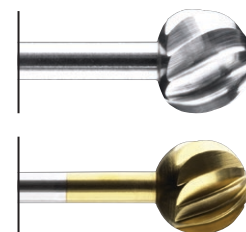

*Single use only

Visao[®]

Diamond Burs

Round Coarse Diamond Burs

6.0 – 1.0 mm Diameter

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	MATERIAL	PART NUMBER	/ BOX
	6.0	64	Stainless steel	31116058	1
	6.0	71	Stainless steel	<u>31116065</u>	1
	5.0	64	Stainless steel	31115059	1
	5.0	71	Stainless steel	<u>31115066</u>	1
	4.0	69	Stainless steel	31114065	1
	4.0	72	Stainless steel	<u>31114068</u>	1
	4.0	78	Stainless steel	<u>31114075</u>	1
	3.5	71	Stainless steel	<u>31113568</u>	1
	3.0	72	Stainless steel	<u>31113069</u>	1
	3.0	78	Stainless steel	<u>31113075</u>	1
	2.5	71	Stainless steel	<u>31112569</u>	1
	2.5	77	Stainless steel	<u>31112575</u>	1
	2.0	72	Stainless steel	<u>31112070</u>	1
	2.0	77	Stainless steel	<u>31112075</u>	1
	1.5	71	Stainless steel	<u>31111570</u>	1
	1.5	76	Stainless steel	<u>31111575</u>	1
	1.0	72	Stainless steel	<u>31111071</u>	1
	1.0	76	Stainless steel	<u>31111075</u>	1



Visao[®]

Diamond Burs

Round Fine Diamond Burs

6.0 – 0.5 mm Diameter

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	MATERIAL	PART NUMBER	/ BOX
	6	64	Stainless steel	31136058	1
	6	71	Stainless steel	<u>31136065</u>	1
	5	64	Stainless steel	31135059	1
	5	71	Stainless steel	<u>31135066</u>	1
	4	69	Stainless steel	31134065	1
	4	72	Stainless steel	<u>31134068</u>	1
	4	80	Stainless steel	<u>3121584</u>	1
	3.5	71	Stainless steel	<u>31133568</u>	1
	3	72	Stainless steel	<u>31133069</u>	1
	3	79	Stainless steel	<u>31103075</u>	1
	2.5	71	Stainless steel	<u>31132569</u>	1
	2.5	78	Stainless steel	<u>31102575</u>	1
	2	72	Stainless steel	<u>31132070</u>	1
	2	78	Stainless steel	<u>31102075</u>	1
	1.5	71	Stainless steel	<u>31131570</u>	1
	1.5	77	Stainless steel	<u>31101575</u>	1
	1	72	Stainless steel	<u>31131071</u>	1
	1	77	Stainless steel	<u>31101075</u>	1
	0.5	72	Stainless steel	<u>31130571</u>	1
	0.5	77	Stainless steel	<u>31100575</u>	1



Skeeter[®]

Otologic Drill System


The Skeeter Ultra-Lite Oto-Tool handpiece is a low-speed drill utilizing time-tested technology for every otologic operating room. The curved handpiece and malleable burs are designed for delicate ossicular chain reconstruction procedures.

Skeeter Components

PRODUCT	DESCRIPTION	QTY
3055601	Skeeter Ultra-Lite Motor	1
3155615	Skeeter Sterilization Tray	1
3052460	Skeeter Otologic Metric Scale	1
3055607	Standalone Power Foot Control	1

Oto-Flex Tools

Various

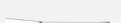PRODUCT	DESCRIPTION		DIA	QTY
NUMBER	NAME AND COLOR	PHOTO	MM	/ BOX
3155627	Crura Saw, Black		0.75	1
3155621	Diamond Knife, Red/White		2.30	1
3155626	Drill, Black		0.60	1
3155625	Perforator, Black		0.50	1
3155623	Perforator, Black		0.35	1
3155628	Trephine, Black		0.80	1

Skeeter Drill System Key Features:

- ① Slender, lightweight, low-speed drill system designed for ossicular reconstruction
- ② Shaft is angled 15° to maximize visualization during drilling up to 16,000 rpm
- ③ Color-coded burs to identify diameter size
- ④ Powered by IPC[®] System or battery-powered footswitch



Skeeter[®]

Cutting Burs

Cutting Burs

Carbide Oto-Flex

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	COLOR	PART NUMBER	/ BOX
●	2.3	98	Red	3155632	1
●	1.8	98	Brown	3155639	1
●	1.4	98	Gray	3155634	1
●	1.0	98	Orange	3155631	1
●	0.8	98	Yellow	3155638	1
●	0.7	98	Green	3155637	1
●	0.6	98	Blue	3155636	1
●	0.5	98	Violet	3155635	1



Skeeter®

Diamond Burs

Diamond Burs

Diamond Oto-Flex

DIAMETER		LENGTH	DESCRIPTION		QTY
SHAPE	MM	MM	COLOR	PART NUMBER	/ BOX
●	2.3	98	Red/white	3155642	1
●	1.8	98	Brown/white	3155645	1
•	1.4	98	Gray/white	3155643	1
•	1.0	98	Orange/white	3155641	1
•	0.8	98	Yellow/white	3155648	1
•	0.7	98	Green/white	3155647	1
•	0.6	98	Blue/white	3155646	1



For further information, please call Medtronic ENT
at 800.874.5797 or 904.296.9600. You may also consult
our website at www.MedtronicENT.com.

Medtronic ENT

6743 Southpoint Drive N
Jacksonville, FL 32216
USA
www.MedtronicENT.com
Toll free: (800) 874-5797
Fax: (800) 678-3995

International Telephone Numbers

Adriatic Region	385-1-488-1120	Italy	39-02-24137-324
Australia	1-800-668-670	Japan	81-3-6430-2017
Baltic Region	37-1-67560226	Korea	82-2-3404-3600
Belgium	32-2456-09-09	Lebanon	961-1-370-670
Canada	1-800-217-1617	Luxembourg	32-2456-09-09
China	86-21-50800998	Netherlands	31-45-566-8800
Czech Republic	420-2-9657-9580	Poland	48-22-4656900
France	33-470-679-800	Russian Fed.	7-495-580-73-77
Germany	49-2159-8149-209	Singapore	65-6776-6255
Greece	30-210-67-79-099	South Africa	27-11-466-1820
Hong Kong	852-2919-1312	Spain	34-91-625-05-40
Hungary	36-30-5052987	Taiwan	886-2-2183-6000
India	91-22-26836733	UK	44-1923-205-166
Israel	972-9-972-4400	USA	1-904-296-9600

